

WELCOME


The Enovate Medical Encore EcoFlex Mobile EHR Workstation was designed to set a new standard in quality. Enovate Medical's goal is to provide a workstation that is built right, ready for years of use, and backed by a commitment of exemplary service and support. Thank you for purchasing the Enovate Medical Encore EcoFlex Mobile EHR Workstation.



CONTENTS


WORKSTATION FEATURES	3
WORKSTATION FEATURES	4
POWERING ON THE WORKSTATION	6
ONBOARD CHARGING	7
ELECTRONIC LOCKING DRAWER (OPTIONAL)	8
SIGHTLINE™ MONITOR ARM	9
LOCKING CASTERS	10
MOBIUSPOWER® SYSTEMS	11

WORKSTATION FEATURES

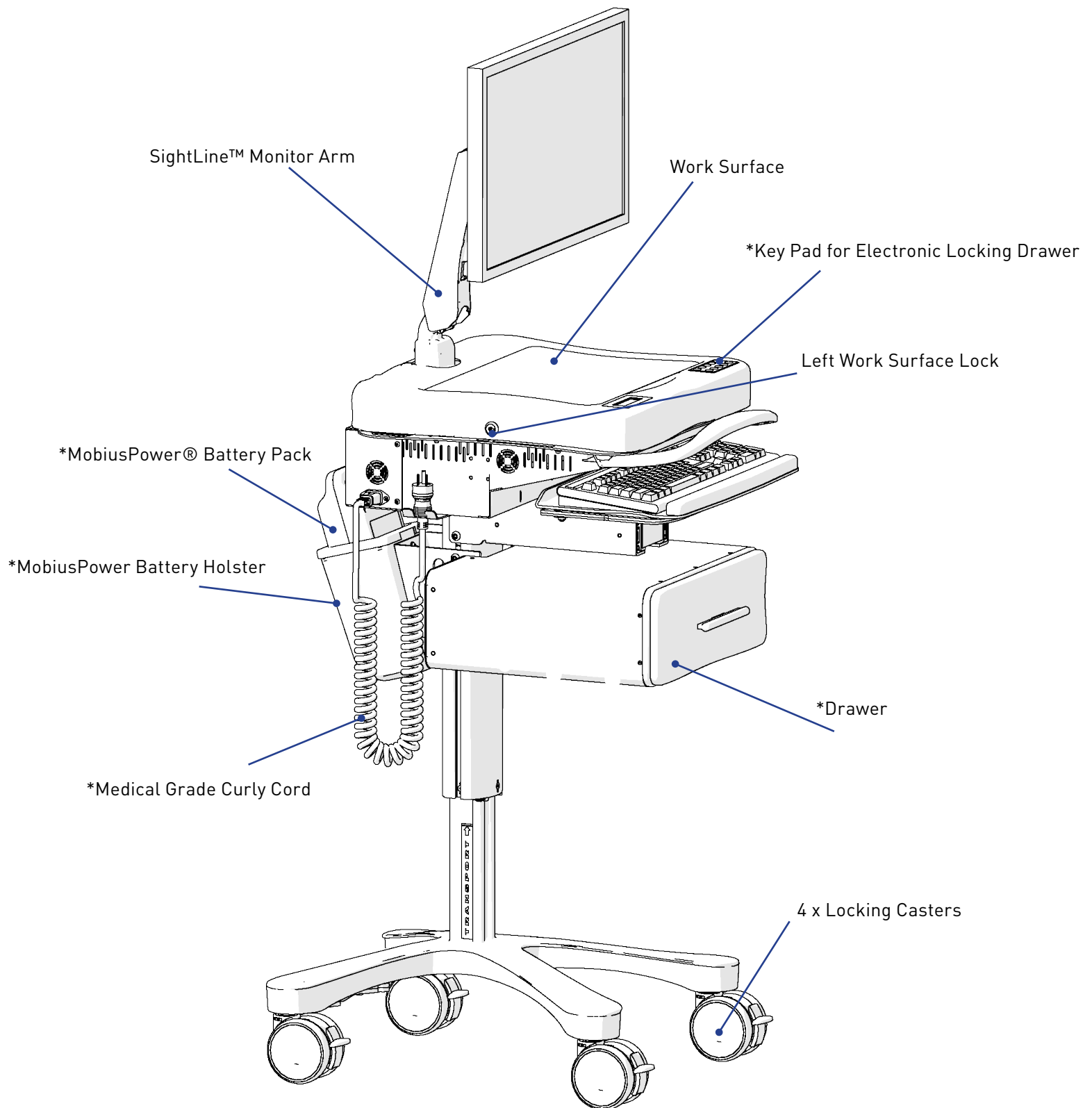

*OPTIONAL ITEM DEPENDENT UPON MODEL


LCD MESSAGE CENTER

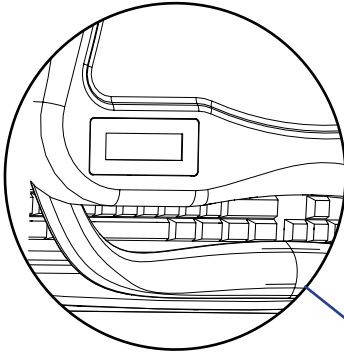

TILTING KEYBOARD


Grip the keyboard tray with both hands and raise upward to adjust for ergonomic comfort.

LEFT AND RIGHT PULL OUT MOUSE PADS

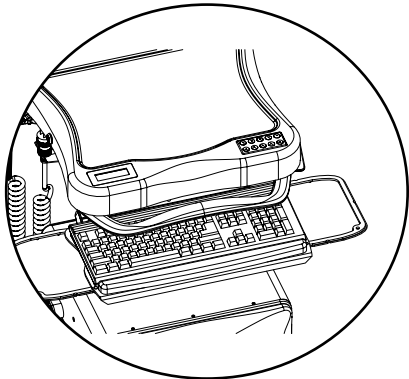

EASY RELEASE CORD


To prevent cord damage if accidentally moved while plugged in.

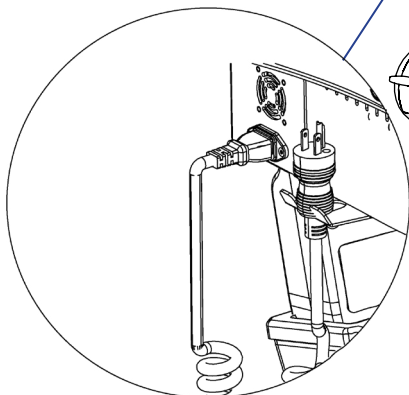

LEFT AND RIGHT SIDE WORK SURFACE LOCKS (right side shown only).


HEIGHT ADJUSTMENT LEVER

Position your hands to hold the work surface steady and grip the height adjustment lever. Pull the lever upward to raise or lower to desired position and release the lever once the desired position is reached.

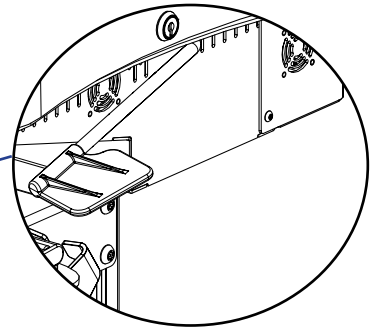


EXTERNAL RESTART PC BUTTON

Depress momentarily to restart PC. Press and hold for >than 10 seconds to restart the workstation.

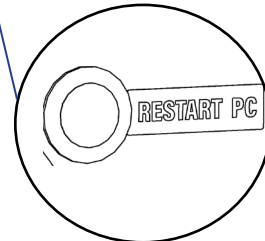


4 LOCKING CASTERS


Press down to lock
Push up to unlock

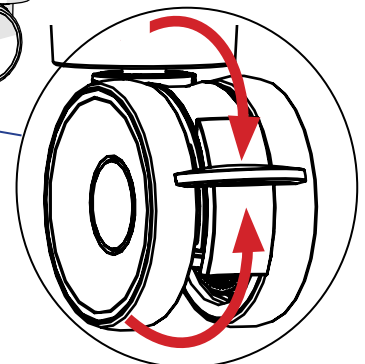


To begin, insert a fully charged MobiusPower Battery Pack or if your workstation is corded plug the cord in to an electrical outlet. The power on indicator will light up blue.

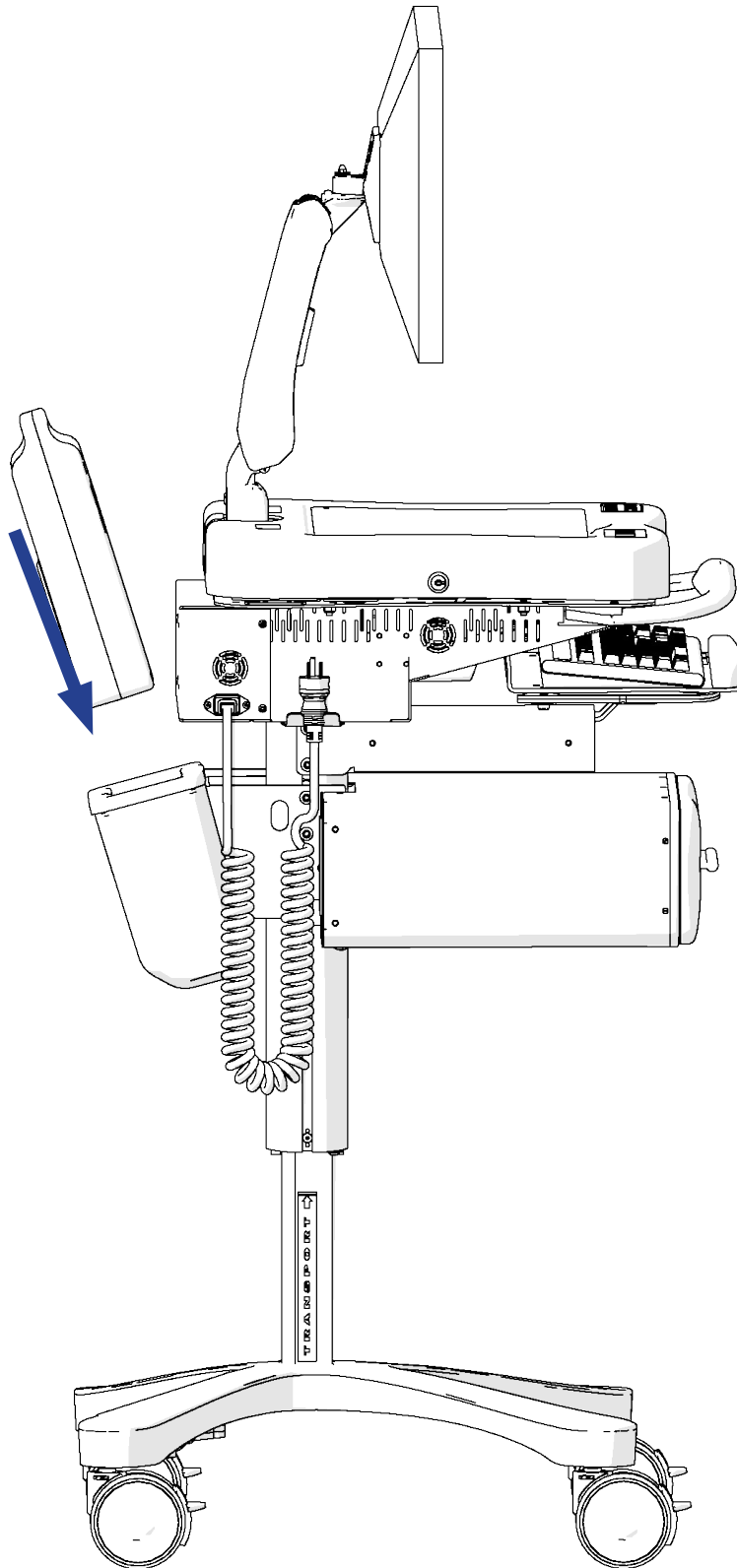


As the Encore EcoFlex™ is powering on, you will see the following screens in sequence appear on the LCD Message Center Screen:

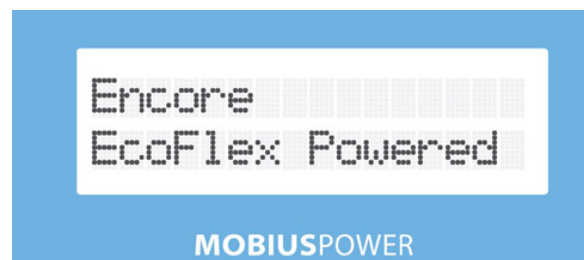


Powering on

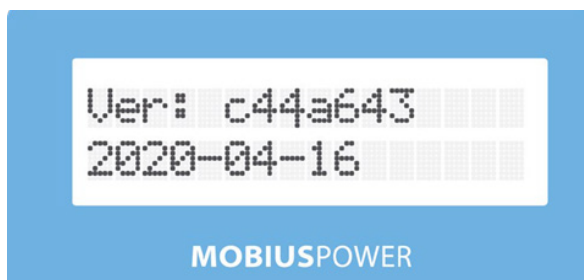


System Version Number and System Version Date

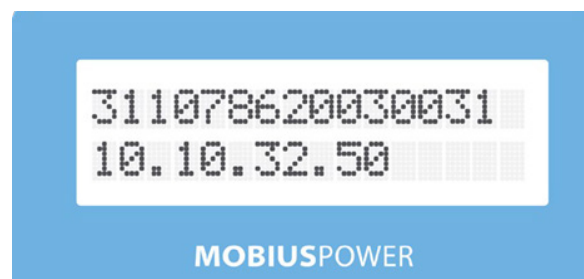

Power System Serial Number and Workstation IP Address

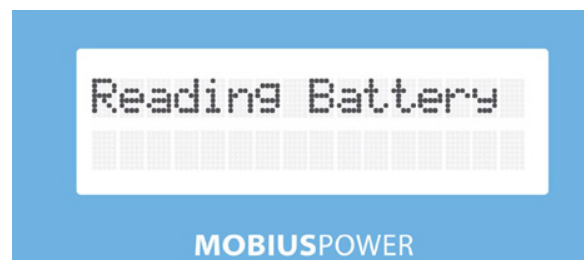

Reading the Mobius Battery

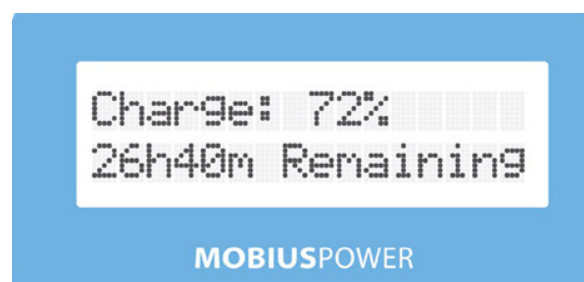


Battery Charge Level and Time Remaining

IMPORTANT INFORMATION


The Encore EcoFlex power system does not directly communicate with any integrated peripheral, like the PC, laptop, Thin Client, etc. The EcoFlex power system does not support our legacy MobiusSD or PulseTech Connect applications.

The Encore EcoFlex MobiusPower Plus can charge the Mobius battery while the battery is in the holster. To begin charging your Encore, plug the 3-prong power cable into a standard 110-120VAC grounded AC power outlet. You will observe the follow messages on the LCD when the AC cord is connected.


Please unplug the workstation from the AC Power outlet prior to moving it.

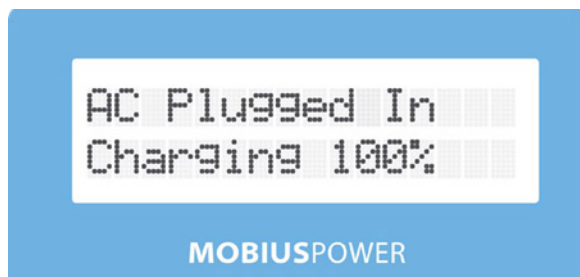


When AC power is connected, and the battery is fully charged

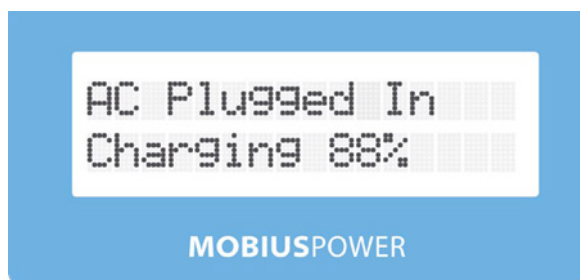

When AC power is connected, and the battery is not fully charged

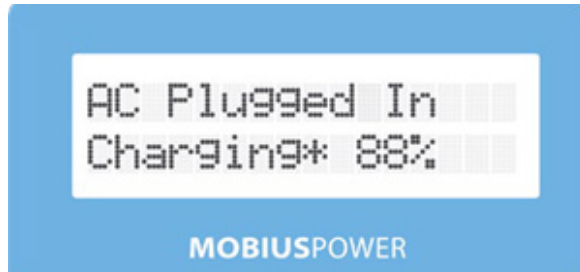

If an asterisk is present, as in, "AC Plugged In, Charging* xxx%," the charger is enabled, but is cooling off for 10-30 minutes

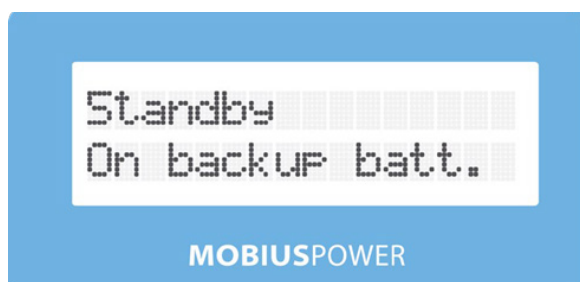

STANDBY MODE

The Encore EcoFlex Mobius Power plus workstation is designed to enter Standby Mode when the battery is near depletion (1% charge or less). In Standby Mode the LCD back lights will be off. All peripherals powered by the workstation, including the PC and the monitor, will also be powered off in Standby Mode. The blue LED power indicator on the power button will remain illuminated while the workstation is in Standby Mode.

Replace the Mobius battery to bring the workstation out of Standby mode and resume normal functionality.

If the workstation is an Ecoflex Encore Plus, you have the option of powering the workstation via AC and charging the battery as well. Please refer to "Charging" for additional details regarding charging the EcoFlex Encore Plus.

AUTOMATIC UPDATES


Your Encore Ecoflex workstation will receive periodic updates. The automatic update window is between 2:00AM - 4:00AM EST. The workstation will reboot after installing the update which will indirectly cause the PC to power off and then back on.

ELECTRONIC KEY PAD




Please follow these instructions to open the workstation drawer if your workstation is equipped with an electronic locking drawer.


1. Using the keypad, enter the assigned pin.

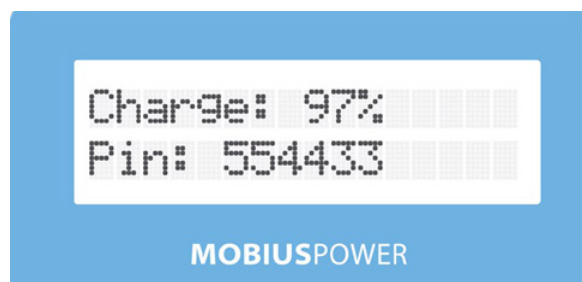


2. If you enter an incorrect number, you can press the "Clear" button on the keypad to clear code one digit at a time.

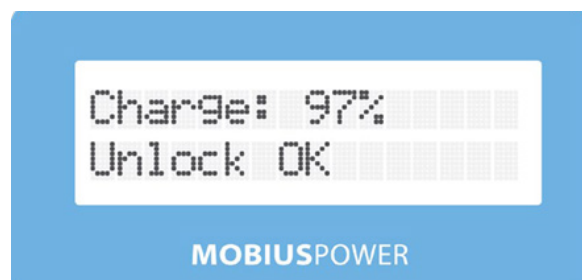

3. If you entered the correct key combination, the LCD will state "Unlock OK" and the drawer will open.

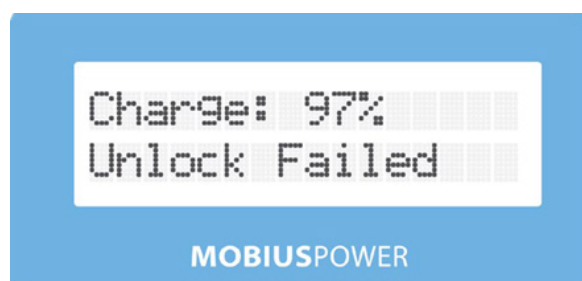

4. If you enter an incorrect key combination, the LCD will read "Unlock Failed"

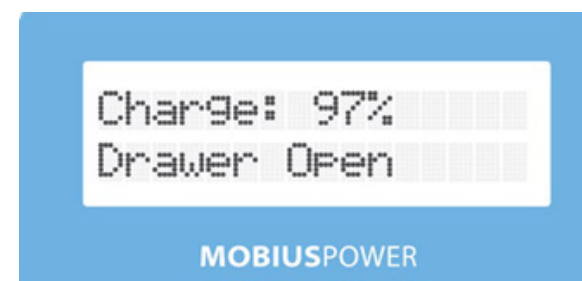


If the drawer remains open for longer than the preset time, the workstation will emit an audible alert and the LCD will read "Drawer Open"


To manually lock the drawer, ensure the drawer is closed, then press the "Lock" button on the keypad. To mute the Warning Alerts, press the "Mute" button on the keypad.

SightLine Monitor Arm Positions for ergonomic comfort, enhanced patient engagement and safe travel.



Upright Position

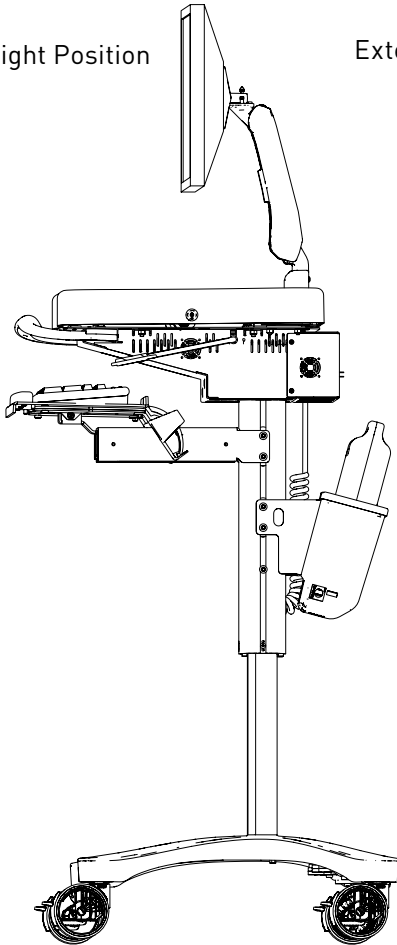


Extended Position

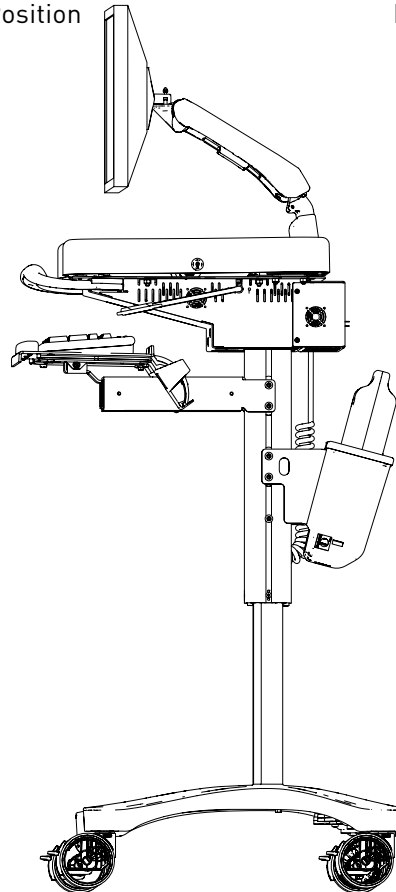

Reclined Position

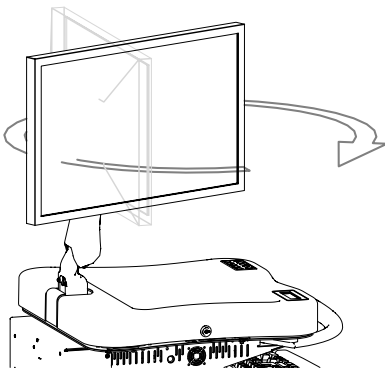
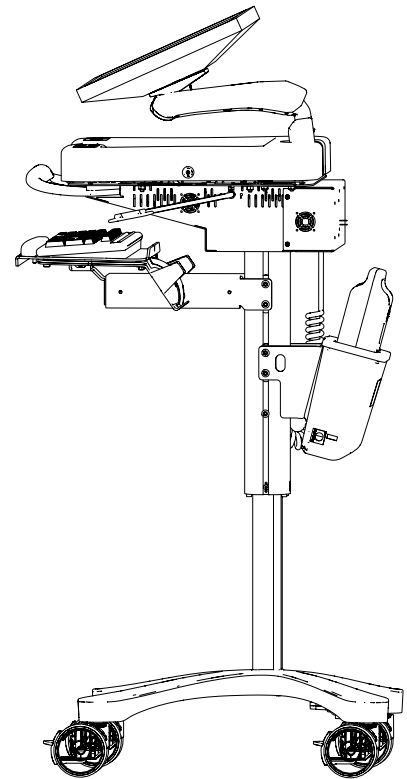


Monitor rotates 80 degrees.

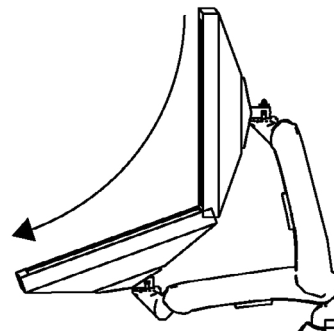

For safe travel, lower the Sightline Monitor Arm to the reclined position.

Press down on caster to lock, push up to unlock.

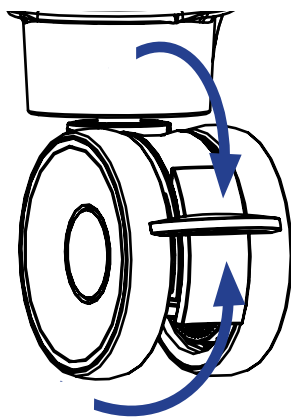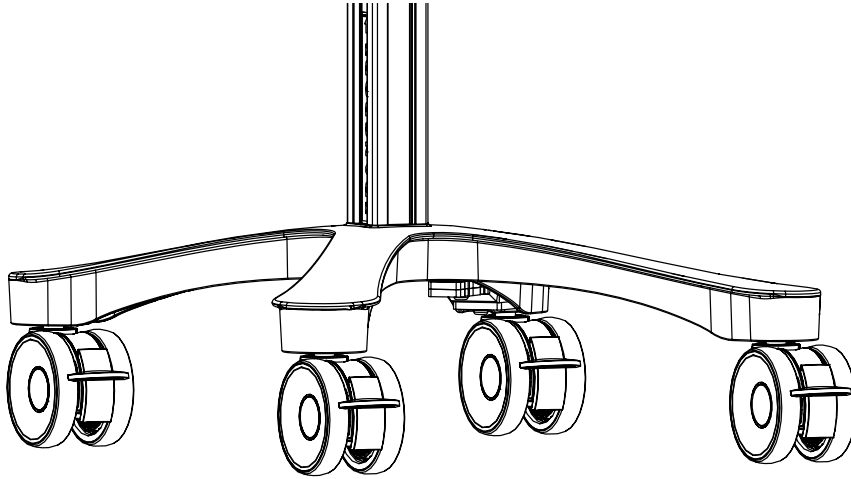


Press down to lock

Push up to unlock


CHARGING BAY

Includes 2-Battery or 4-Battery Charging Bays. Once the charging bay is placed and plugged into an outlet, slide a battery pack into any open slot making sure the FLASHING GREEN indicator light turns on. The charge cycle is complete when the indicator light is illuminated SOLID GREEN.

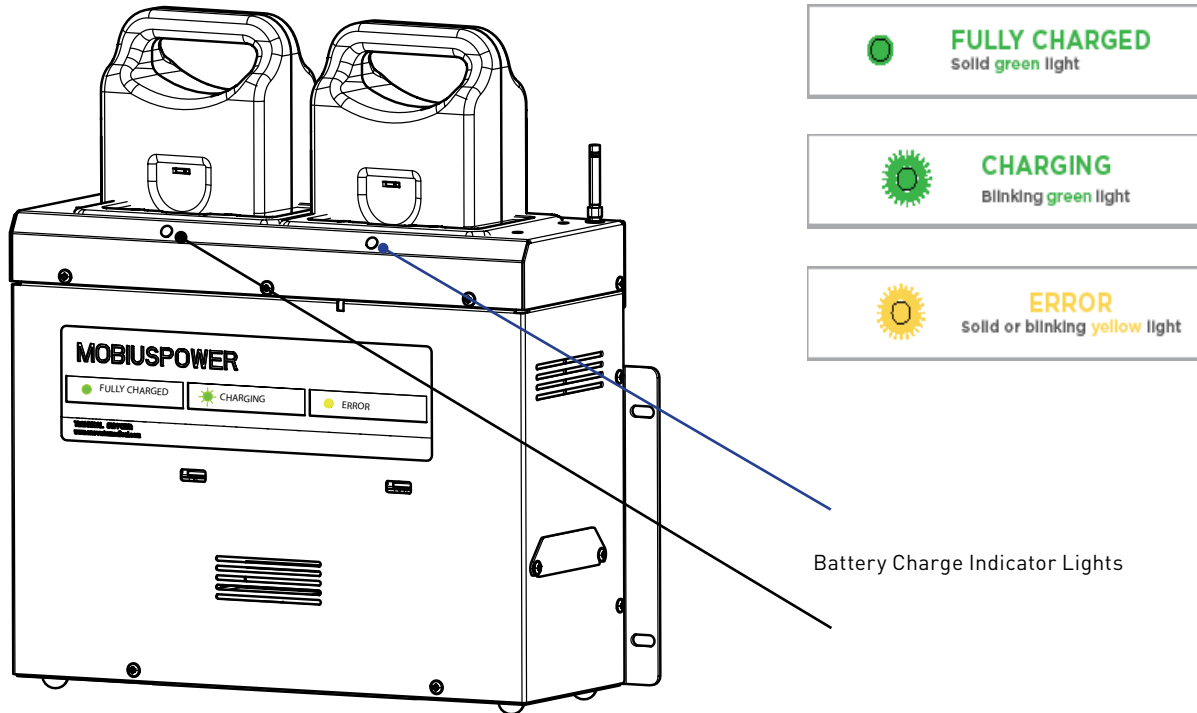

Each Battery Pack has a “fuel gauge” to indicate charge level/capacity.

- You may check a Battery Pack’s charge capacity at any time.
- To check the remaining charge on a Battery Pack, press and release the white circle on the Battery Pack’s fuel gauge display.
- The charge level is indicated in 25% increments by the number of status lights appearing between the E-Empty and F-Full.
- After 3 seconds, the status lights will turn OFF.

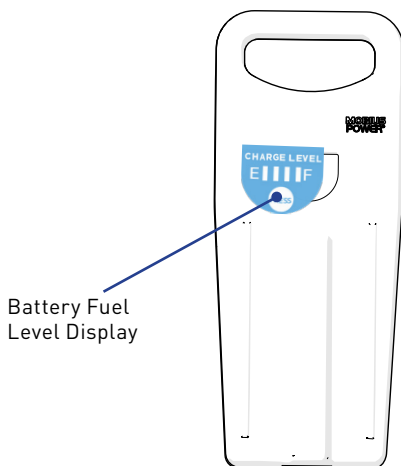

Charge Capacity		LED 1	LED 2	LED 3	LED 4
E █ █ █ █ F	0% — 10%	FLASH	OFF	OFF	OFF
E █ █ █ █ F	11% — 25%	ON	OFF	OFF	OFF
E █ █ █ █ F	26% — 50%	ON	ON	OFF	OFF
E █ █ █ █ F	51% — 75%	ON	ON	ON	OFF
E █ █ █ █ F	76% — 100%	ON	ON	ON	ON

For additional information or questions regarding this product, please contact us at:

1-888-909-8930

Enovate Medical
1152 Park Avenue
Murfreesboro, TN 37129