

enovate



M E D I C A L

QUICK START GUIDE




STARTING WORKSTATION

IMPORTANT - STEPS 1 AND 2 **MUST** BE FOLLOWED IF THIS IS THE FIRST TIME THE UNIT IS BEING TURNED ON OR IF THE UNIT HAS BEEN INACTIVE FOR A PROLONGED PERIOD OF TIME RESULTING IN COMPLETE POWER DRAINAGE.

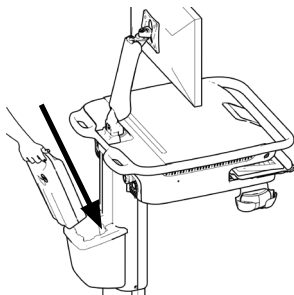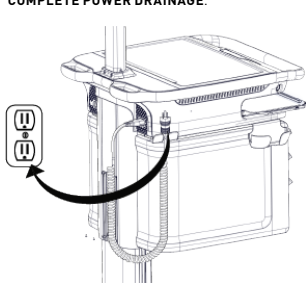

1

Plug the workstation into an electrical outlet to charge, or if the unit is a Mobius workstation, insert a fully charged Mobius battery.

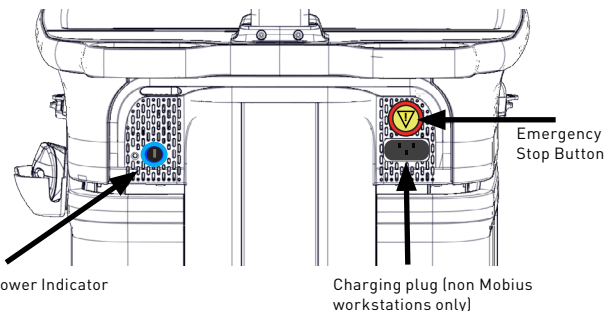



2

The power indicator light is located in the rear of the workstation. Plug in the workstation or insert a fully charged Mobius battery, you will hear a confirmation beep when the workstation is powered on. **IMPORTANT:** Check to ensure that the emergency stop button is not pushed in. If it is, twist left to reenale the actuator power. (The emergency stop button interrupts the up/down movement of the workstation.) If this is the initial deployment see product manual.

LOGGING INTO THE WORKSTATION

Once the workstation is powered on, the login touch screen will appear (this takes approximately 30 seconds). The touch display is designed to turn off to save energy, please tap the display or bump the workstation to turn it on.


" There are three ways to login to your workstations, see step 3.

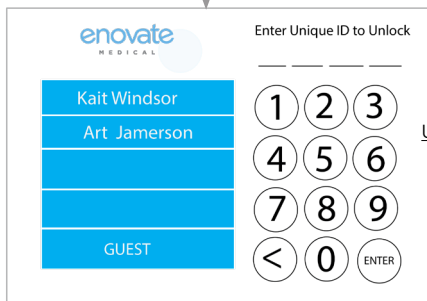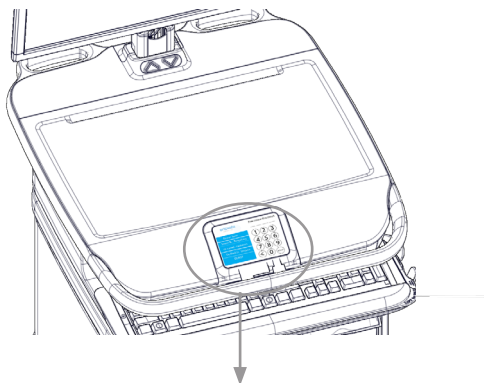**There are three ways to login to the Envoy****3**

- 1. Key FOB Login:** Ensure your key FOB is powered on, and select your name from the list when it appears. *(Please see your IT department for key FOB login details).*
- 2. Manual Login:** Enter your PIN code. *(Please see your IT department for PIN code details).*
- 3. Guest Login:** Sign in as a "Guest" by selecting "Guest" from the user list.

USER INTERFACE

HOME SCREEN DETAILED VIEW

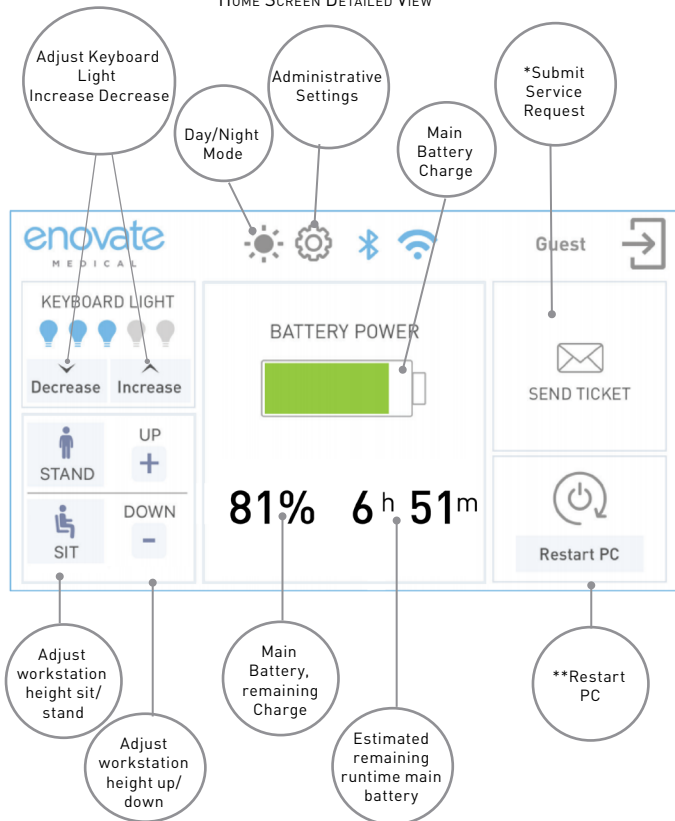



4

There are two ways to adjust the height of the workstation 1) use the +/- icons on the touch display or 2) use the up/down arrow keys located at the base of the monitor arm. Sit/stand is set to factory preset heights, programming for personalization is available, please see the product manual for details. If the workstation has medication bins, please see #5.

* Submit "service request" feature only available on workstations connected to RHYTHM.

** "Restart PC" May be disabled by administration

HEIGHT ADJUSTMENT

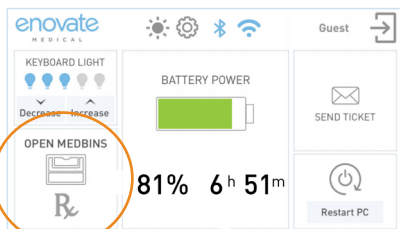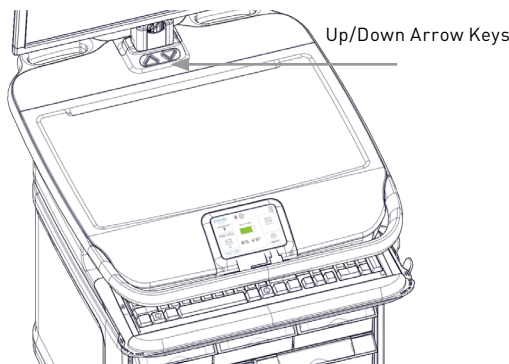

HOME SCREEN DETAIL VIEW - MEDBIN UNIT



IMPORTANT: If your workstation has medication bins, the up/down arrow icons will be replaced with a medbin icon.

5


Medbin Units Only: For workstations with medication bins use the up/down arrow keys on the workstation located at the base of the monitor arm to raise and lower the unit. "Full sit/stand" functionality is available by double pressing the arrow keys. To access medbins tap the RX icon on the touch display and enter your four digit pin. **(See touch display user interface for full details.)**

SIGHTLINE MONITOR ARM POSITIONING

Sightline Monitor Arm Positions for ergonomic comfort, enhanced patient engagement and safe travel.

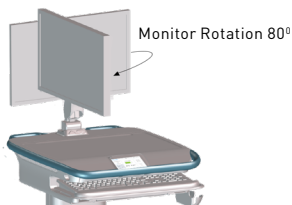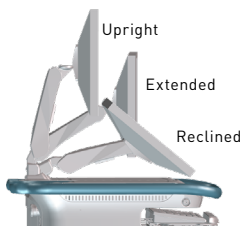

SAFE MOBILITY

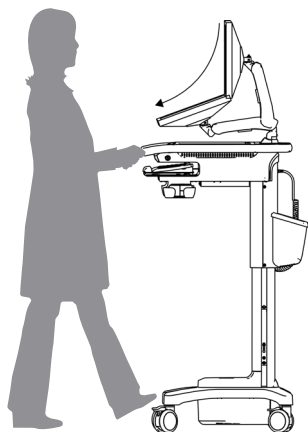

For Safe Travel Lower Sightline Monitor Arm to the reclined position



Press down on brake to lock casters, and push brake up to release.

6

For safe travel, place the sightline monitor in the reclined position. Release brakes to allow movement; Autotrax casters automatically engage for effortless steering.

