

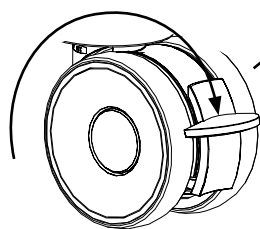


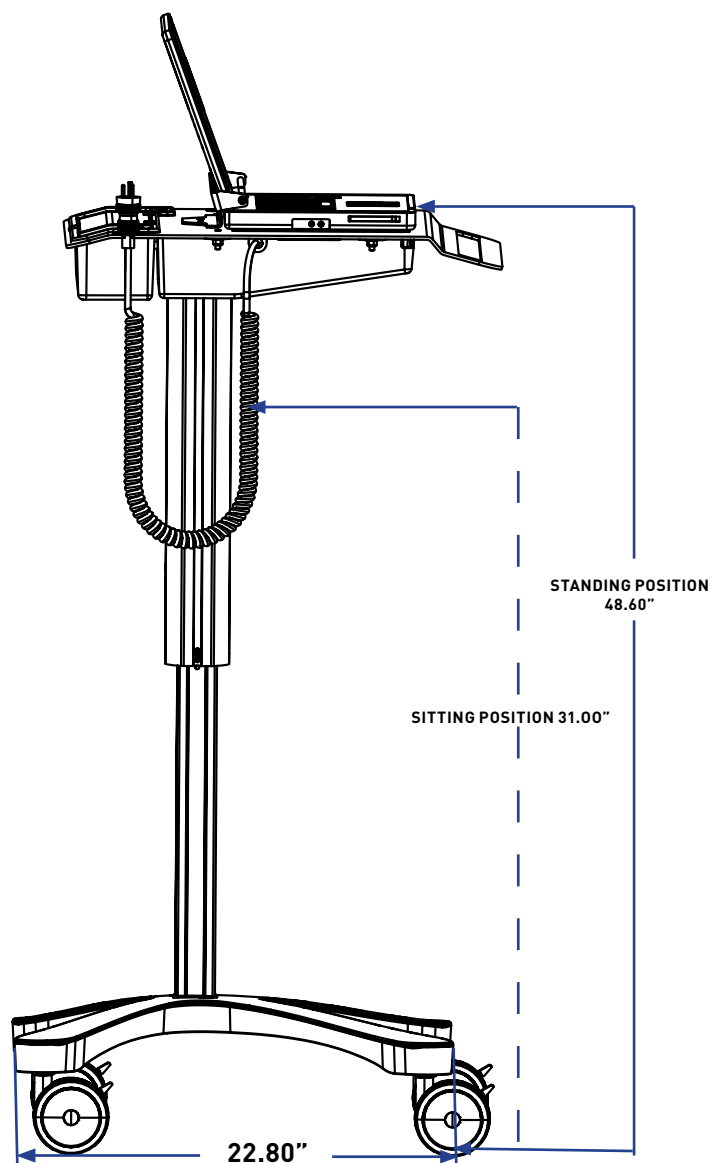

Contents


WELCOME	3
WORKSTATION FEATURES	4
HEIGHT ADJUSTMENT	5
NEMA RECEPTACLE AND ELECTRICAL CORD	6
LAPTOP SECURITY BAR	7
POWERING ON	8
COIL CORD AND NEMA SPECIFICATIONS	9


The Enovate Medical Encore Lite Mobile EHR Workstation was designed to set a new standard in quality. Enovate Medical's goal is to provide a workstation that is built right, ready for years of use, and backed by a commitment of exemplary service and support. Thank you for purchasing the Enovate Medical Encore Lite Mobile EHR Workstation.


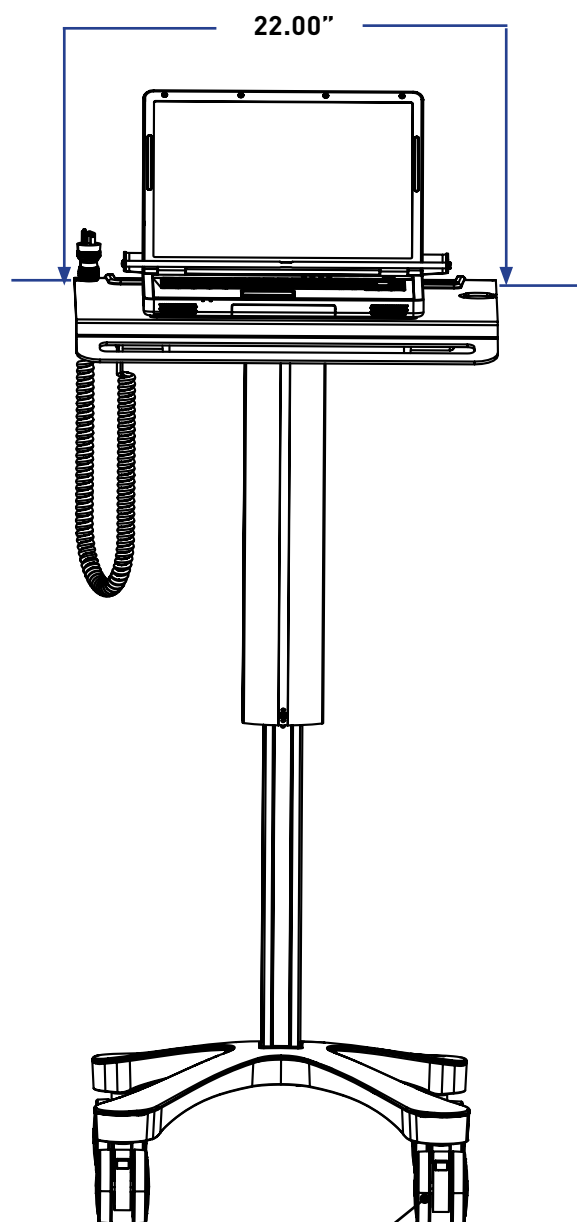


LOCKING CASTERS
Press down on brake to lock the casters, push up on brake to release.


To adjust the height of your workstation lift up on the height adjustment lever to move the workstation up or down.

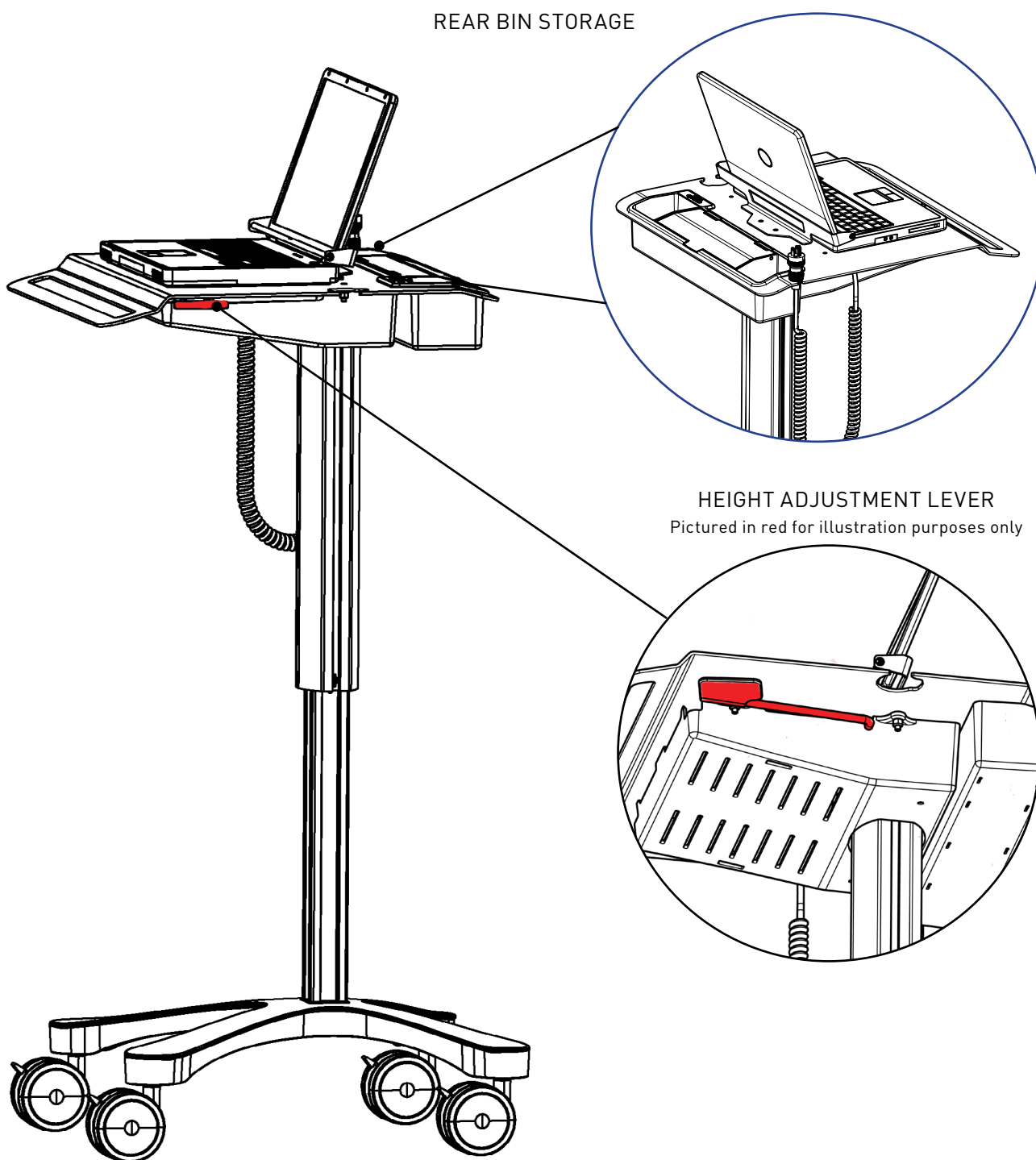

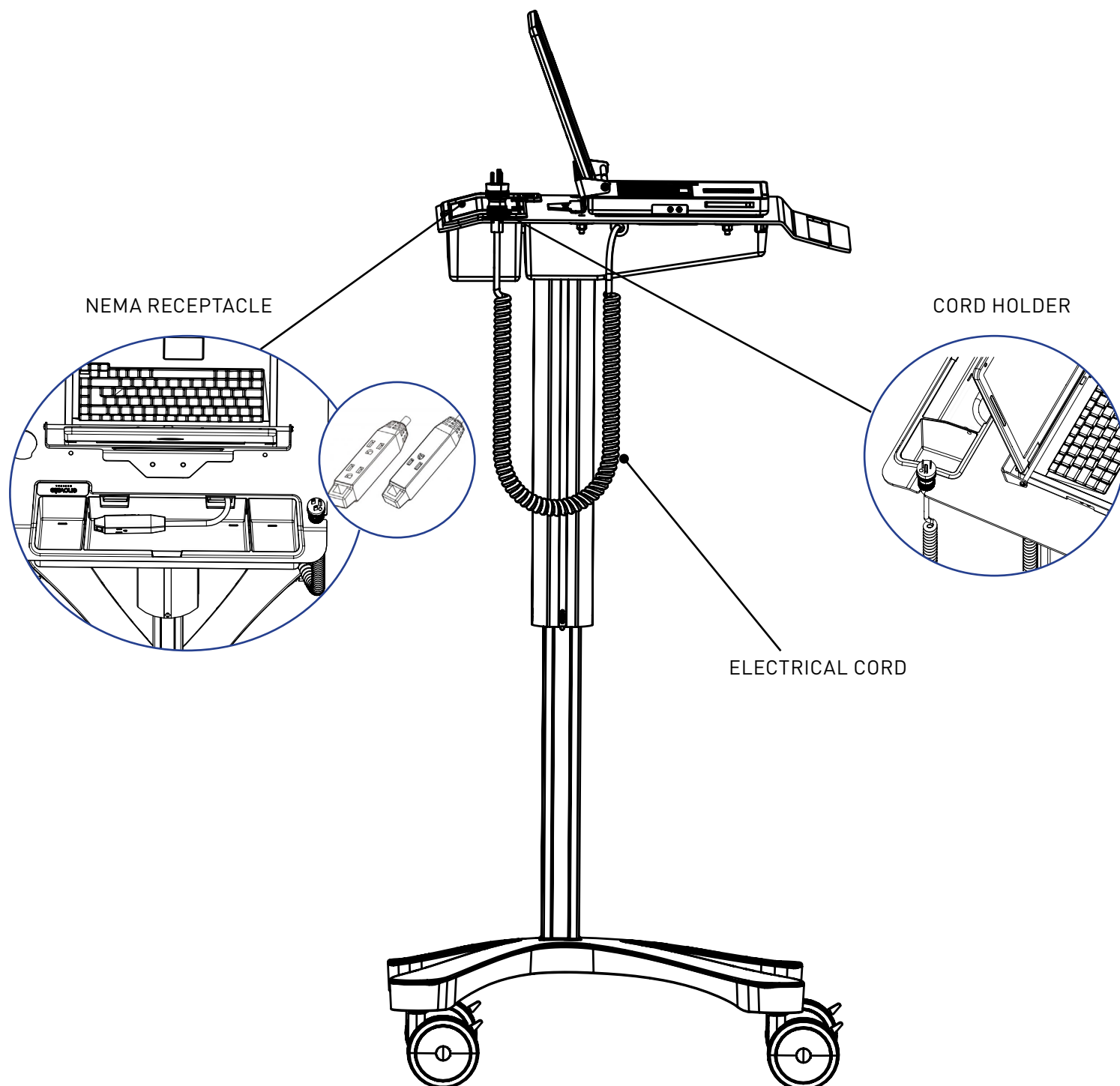

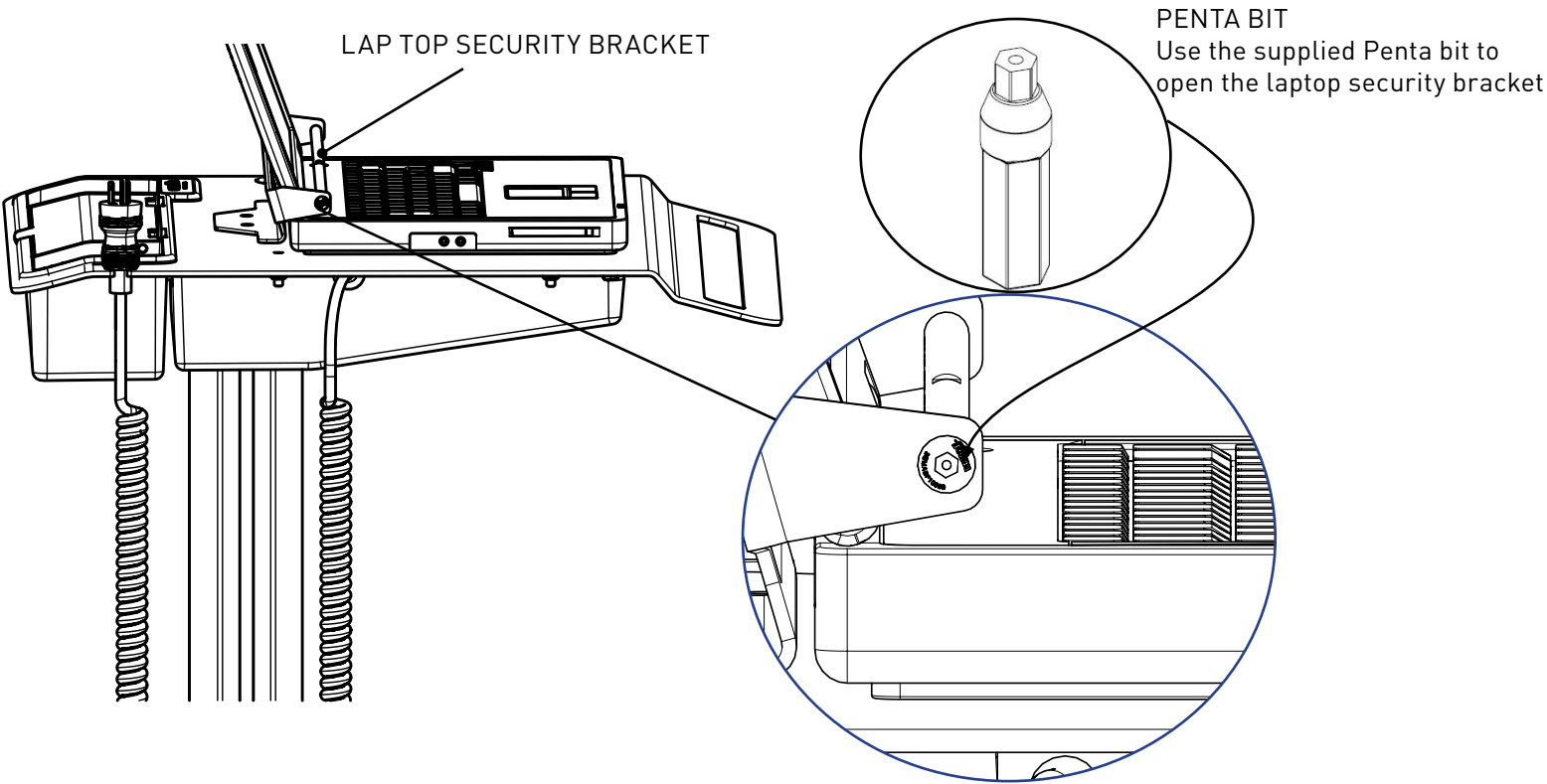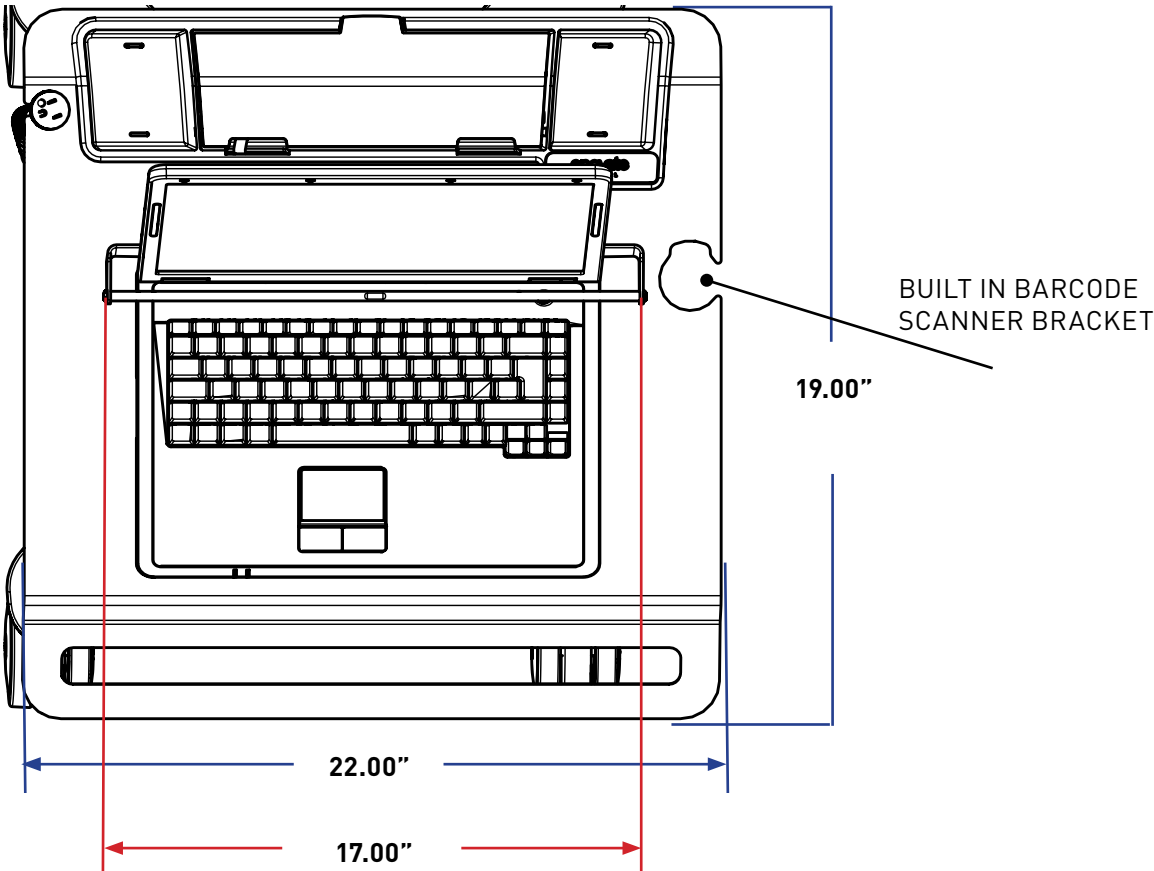

To recharge laptop or power on, plug your workstation into an electrical outlet.

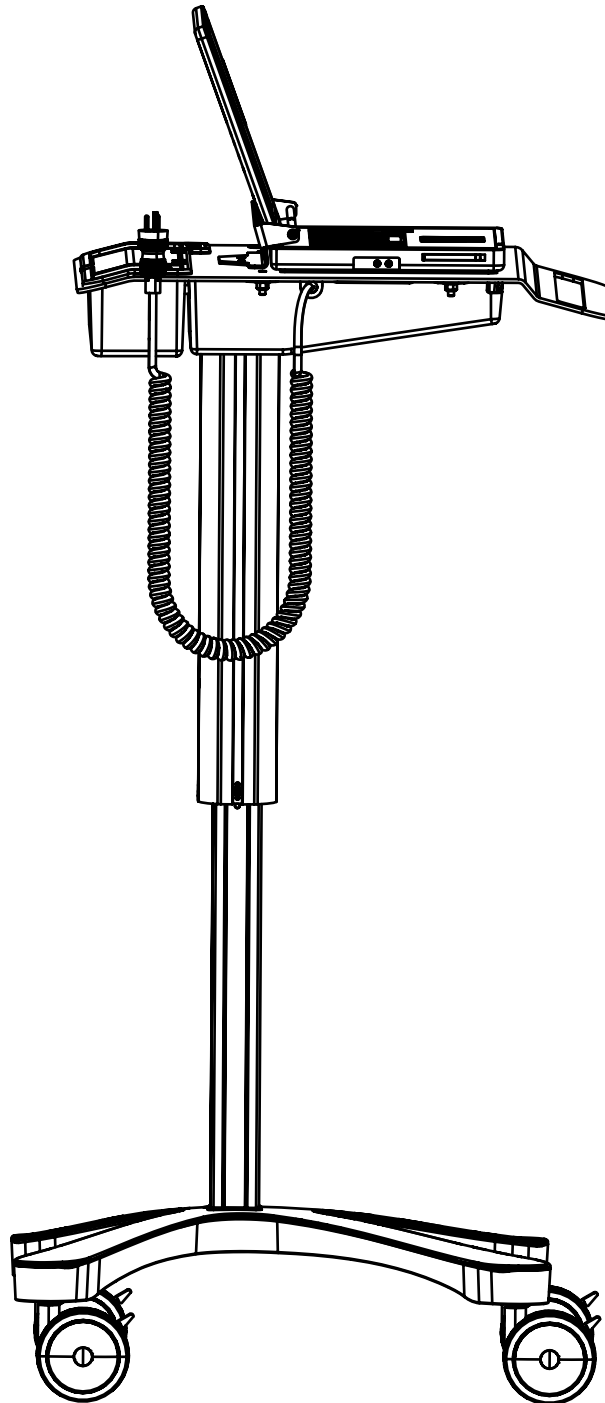

COIL CORD SPECIFICATIONS:

CABLE: SJT 18AWG 3C 105°C (E90165) 300V APPROVAL: UL, CSA

NEMA PLUG: WS-001H-1 15(A) 125(V) APPROVAL: UL, C-UL

IEC 320 (C13) PLUG: 10(A) 125(V) APPROVAL: UL, C-UL

NEMA RECEPTACLE SPECIFICATIONS:

CABLE: SJTW 18AWG 3C 60°C (E90165) 300V APPROVAL: UL, CSA

NEMA RECEPTACLES: WS-116-5 10(A) 125(V) APPROVAL: UL, C-UL

IEC 320 (C14) PLUG: 10(A) 125(V) APPROVAL: UL, C-UL

For additional information or questions regarding this product, please contact us at:

1-888-909-8930

Enovate Medical
1152 Park Avenue
Murfreesboro, TN 37129